

**NANO M.ON**

DISPOSABLE

# Premium CE nano fibre medical face mask



with double filtration efficiency that captures **99,9 % of viruses**, bacteria, mould, dust particles and smog, protecting the user

**Disposable. Risk of contamination when used repeatedly.**

## Instructions for use:

### How to put on a face mask with rubber straps

1. Wash and wipe your hands and remove the face mask from the box.
2. Grasp the reinforced bendable edge upwards.
3. Hold the face mask by the loops and hook it around your ears.
4. Adjust the reinforced edge to the shape of your nose and pull the bottom edge of the mask over your mouth and chin.
5. Do not touch the mask while wearing it.



### How to remove a face mask with rubber straps

1. Wash and wipe your hands.
2. Do not touch the front part of the face mask which is contaminated. Only touch the rubber straps.
3. Grasp both rubber straps and carefully lift and remove the face mask.
4. Dispose of the face mask.
5. Wash your hands.



Manufacturer: Nano Medical s.r.o., member of the Association of the Nanotechnology Industry of the Czech Republic, 5. května 1640/65, 140 00 Praha 4

Exclusive distributor: FERMATA, a.s., Zárubova 1678, 250 88 Čelákovice, Czech Republic, e-mail: [cdf@fermata.cz](mailto:cdf@fermata.cz), telefon: +420 326 901 311, [www.fermata.cz](http://www.fermata.cz), [www.nanomon.cz](http://www.nanomon.cz)

Version: 24 September 2020